EN Prior to use, carefully read the instructions for use.

## GC Tooth Mousse™

TOPICAL CREME WITH BIO-AVAILABLE CALCIUM AND PHOSPHATE For use by or under the supervision of a dental professional in the recommended indications.

**Description**  
GC Tooth Mousse is a water based cream containing RECALDENT™ (CPP-ACP: Casein Phosphopeptide – Amorphous Calcium Phosphate). When CPP-ACP is applied to the oral environment, it will bind to available calcium and soft tissue localising bio-available calcium and phosphate.

**Important:**  
GC Tooth Mousse enhance the effectiveness of CPP-ACP and the favour will help maintain saliva flow. The longer CPP-ACP and saliva are held in the mouth, the more effective the result.

1. **Apply a sufficient amount of bio-available calcium and phosphate** 1. Provides extra protection for teeth. 2. Helps neutralize acid challenges from acidogenic bacteria in plaque. 3. Helps neutralize acid challenges from other internal and external acid sources.

**CONTRAINDICATIONS**  
RECALDENT™ (CPP-ACP) is derived from milk casein. Do not use this material on patients with a proven or suspected milk protein allergy and/or with a sensitivity or allergy to benzate preservatives.

**DIRECTIONS FOR USE**  
**INDICATIONS**  
a. Prior to and following in-office bleaching. b. After ultrasonic, hand scaling or air polishing. c. Following professional tooth cleaning (P.T.C.). d. After application of topical fluoride. e. To provide a topical coating for patients suffering from erosion, xerostomia or Sjögrens syndrome. f. During orthodontic treatment. g. For high risk caries patients. h. For special needs patients.

**Non-tray application**  
1. Custom tray application  
1. Before use rinse the custom tray thoroughly under running water. 2. Apply a sufficient amount of GC Tooth Mousse into the tray and apply to the upper and/or lower teeth. 3. Leave the tray undisturbed in the mouth for a minimum of 3 minutes. 4. Remove the tray. 5. Instruct the patient to use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Instruct the patient to refrain from eating or drinking for 30 minutes following application and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result. 6. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually erode over the next 24 hours. Do not eat or drink for 30 minutes following application.

7. Any residual GC Tooth Mousse in the tray should be rinsed or brushed away from the tooth surface immediately after use.

**II. Non-tray application**  
1. If necessary, remove any excess saliva on the tooth surface with a clean gauze pad. However, it is NOT necessary to dry the teeth with compressed air. 2. Apply a sufficient amount of GC Tooth Mousse to the tooth surfaces using a finger or cotton tip. For difficult areas (between the teeth) use Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse. 3. Leave GC Tooth Mousse undisturbed for a minimum of 3 minutes. 4. Remove the tooth Mousse and use the tongue to spread the remaining GC Tooth Mousse throughout the mouth. Request the patient to hold in the mouth for as long as possible (in addition 1 – 2 minutes) avoiding expectoration and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result. 5. Ask the patient to expectorate and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually erode over the next 24 hours. Do not eat or drink for 30 minutes following application.

**III. Night time application after tooth brushing as recommended by a dental professional**

1. Apply a sufficient amount of GC Tooth Mousse to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean finger or cotton tip. For difficult areas (between the teeth) use Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse. 2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes. 3. Spread your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (in further 1 – 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result. 4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually erode over the next 24 hours. Do not eat or drink for 30 minutes following application.

**IV. Night time application after tooth brushing as recommended by a dental professional**

1. Apply a sufficient amount of GC Tooth Mousse to the upper and lower teeth. A pea size amount for each arch is the minimum required. The material should be applied to the tooth surfaces using a clean finger or cotton tip. For difficult areas (between the teeth) use Interproximal Tooth Cleaning Brush or dental floss coated with GC Tooth Mousse. 2. Leave GC Tooth Mousse on the teeth undisturbed for a minimum of 3 minutes. 3. Spread your tongue to spread the remaining GC Tooth Mousse throughout the mouth. Hold in the mouth for as long as possible (in further 1 – 2 minutes) avoiding expectoration (spitting out) and delaying swallowing. The longer GC Tooth Mousse and saliva are maintained in the mouth, the more effective the result. 4. Expectorate thoroughly and if possible avoid rinsing. Any GC Tooth Mousse remaining on the surface can be left to gradually erode overnight. Do not eat or drink for 30 minutes following application.

**FLAVORS**  
Strawberry, Melon, Vanilla, Mint, Tutti-Frutti

**STORAGE**  
Recommended for optimal performance, store in a cool dry low humidity area away from direct sunlight (8-25°C, 46-47.7°F).

**PACKAGES**  
10 x 40g (2 each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti)  
2. Single Flavor Package  
10 x 40g in each of Strawberry, Melon, Vanilla, Mint, Tutti-Frutti, Vanilla, Strawberry, Melon, Tutti-Frutti, Mint  
3. Retail Package  
1 x 40g tube in each flavor

**NOTE**  
GC Tooth Mousse with its unique CPP-ACP formula was developed at The School of Dental Science, The University of Melbourne, Victoria, Australia. RECALDENT is a trademark used under licence. CPP-ACP technology has related patents or patents pending in Australia, NZ, Europe, Canada, and USA. **CALCIUM**  
1. Always replace cap after use and use any residual paste or moisture under the tube neck or inside the cap. 2. In case of contact with eyes, flush immediately with water and seek medical attention. 3. In case of spillage on clothes, wash up with water. 4. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzate preservatives, or to any other component of the product such as a flavoring agent. In this event, discontinue the use of the product and refer to a physician. **Other products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at:** <http://www.gceurope.com>

**CAUTION**  
1. Always replace cap after use and use any residual paste or moisture under the tube neck or inside the cap. 2. In case of contact with eyes, flush immediately with water and seek medical attention. 3. In case of spillage on clothes, wash up with water. 4. If any angioedema symptoms are experienced, this may indicate sensitivity or allergy to the benzate preservatives, or to any other component of the product such as a flavoring agent. In this event, discontinue the use of the product and refer to a physician. **Other products referenced in the present IFU may be classified as hazardous according to GHS. Always familiarize yourself with the Safety Data Sheets available at:** <http://www.gceurope.com>

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