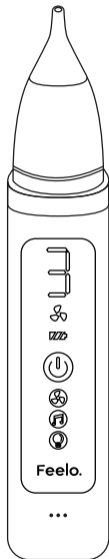
Feelo.

BABY NASAL ASPIRATOR

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The instruction applies to model with the following number: FF.AS.010.WHT.1

SET CONTAINS:

- 1 nasal aspirator
- 3 different silicon tips
- 1 clip
- 1 case
- 1 USB charging cable.

DESCRIPTION

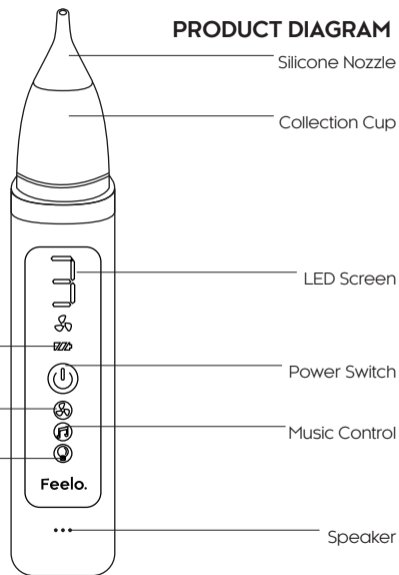
The Feelo electric nasal vacuum aspirator was created for the youngest. It enables easy, fast and hygienic removal of the nasal discharge. Especially important because infants and young

children are not able to clean the nose on their own. The nasal aspirator removes both sparse and dense secretions. Brings immediate relief in breathing. Prevents the development of infection, cough and allergy symptoms. Device is designed to help children from the first days of life up to the age of 12 .

MAIN CHARACTERISTIC:



- 3 degrees of suction (gentle, moderate, strong). Adapts to needs of different age groups. Maximum suction force is 68 kPa.
- Low noise levels of the device allow for the efficient cleaning of the baby's nose. Even while the baby sleeps.
- Soft silicone tips are delicate and do not irritate the mucous membrane of the nose. Made of safe materials. Free from BPA and latex.

- Built-in battery provides up to 2-3 hours of continuous operation on a single charge in gentle mode (first suction level).
- 10 cheerful melodies and lights make cleaning the nose more comfortable and less stressful for the child.
- The Feelo nasal aspirator is lightweight and easy to use, so it will work perfectly at home and on the go.

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	 CALABASH TIP	 FLATING TIP	 BEVEL TIP
Recommended age (years)	0-2	0+	0+
Use	Recommended for cleaning the nose of newborns. The calabash tip makes it easier to extract the discharge while also preventing the tip from being pushed too deep into the nose.	The flating tip can be inserted a little further into the nose. It works especially well with slightly older children.	Bevel Tip makes it easier to remove discharge adjacent to the inner walls of the nose.

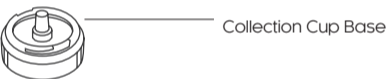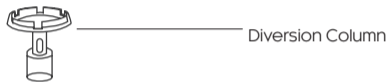BUTTON DESCRIPTION

Press and hold the button to turn the device on/off. A short press of the button starts/stops the nasal aspirator.

A short press of the button switches between modes. The first press selects the soft mode. Next press selects moderate mode. Following press selects strong modes.

Press briefly for the first time to turn on the tune; the default volume is set at 50%. Press briefly for the second time to increase the volume to 100%. Press briefly the third time to stop playing the melody. Repeat the sequence to turn on another tune.

Press briefly to turn the color backlight on/off.

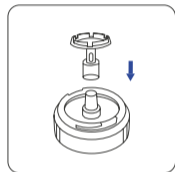
CHARGING

Connect the product via a USB cable to the power supply. The nasal aspirator should be fully charged before first use. Charging cycle takes about 3h. The battery indicator flashes while the aspirator is charging and lights up continuously when the device is fully charged. To extend battery life charge the device when signaled. In case of sparse use charge the battery up to 100% every 2 months.

Important Note: Make sure you are charging with the appropriate voltage (5.0V) and current (1.0A). Make sure that power supply and cable are properly connected to the charging port. Use of a power supply with inconsistent parameters can lead to malfunction of the device. Failure to comply with this principles may result in damage to the device as well as void of the warranty.

USAGE


It is recommended to sterilize the silicone tip and collection cup prior to use. It is recommended to sterilize in boiling water or steam.


1. Install the diversion column on the collection cup seal into the collection cup base.

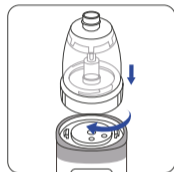
2. Screw the collection cup seal into the suction chamber base.

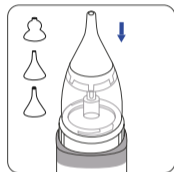
3. Screw the collection cup into the main body and connect with the body.

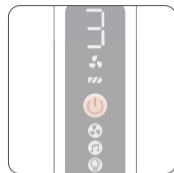4. Select desired nozzle and attach it to the collection cup mask.

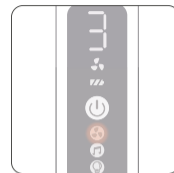
5. Long press the power button to start the device (Long press again to turn off the device).


6. Short press the suction adjustment button to adjust to the required suction intensity.

- 1 press - gentle mode
- 2 presses - moderate mode
- 3 presses - strong mode

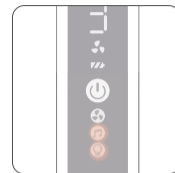To test the strength of the device, you can put the tip of the aspirator against the top of your hand.

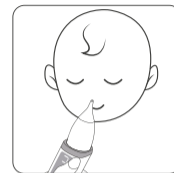
7. Short press the music button or light button to attract your baby. These functions are optional.


8. Place the suction nozzle in the nasal cavity to clean the baby's snot. Suck the secretion from each nasal opening for a few seconds. Repeat the action as needed.

IMPORTANT NOTICE:

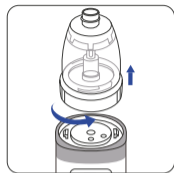
- While using the nasal aspirator, make sure child's mouth is open. This will ensure proper air circulation in the nasal cavity while avoiding discomfort caused by pressure imbalances.
- Do not allow your child to hold his breath, close his mouth or swallow while using the nasal aspirator. Otherwise pressure imbalance may occur.
- When using the device, keep your child in a seated position. Nostrils facing downwards. Do not tilt the child's head.

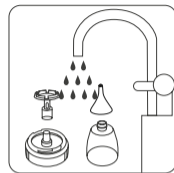9. After cleaning, please turn counter-clockwise to remove the collection cup.

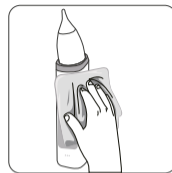10. After removing the collection cup parts, please rinse them in time (Note: the body is not washable).

11. After cleaning, please dry the machine surface with a soft cloth and keep it in a cool place to avoid direct sunlight.

Note: If the baby's nasal cavity snot is dry, please soften it first with saline nasal spray and then use the nasal aspirator. This will achieve better effects.

CLEANING AND MAINTENANCE

- Turn off the device before cleaning nasal aspirator.
- After each use, remove the silicone nozzle and collection cup. Empty the collection cup. Wash all parts with detergent under running water.
- Make sure all parts of the device are clean and dry before reassembling.
- The silicone nozzle and collection cup can be sterilized in boiling water or steam but not in a microwave oven.
- The body of the device is not waterproof. Do not soak or wash it in water or clean in high temperature. Use soft, damp cleaning cloth. Wipe to dry.
- If water enters the air inlet, it should be shaken out or blow-dried. Otherwise it may affect the air pump in the device.
- Store in a dry and cool place. Avoid storage in a damp environment. Avoid contact with sunlight.

TROUBLESHOOTING

PROBLEM	SOLUTION
The nasal aspirator does not turn on or turns off automatically after a few seconds	<ul style="list-style-type: none">• Charge the nasal aspirator. Connect the power cord and charge the device through the power socket. Charging cycle takes about 3h.• If the device does not turn on even though it was recently charged, contact the service.
Poor suction power/ The nasal aspirator does not remove secretions well	<ul style="list-style-type: none">• A nasal aspirator removes pollutants using airflow. Please do not fully align the mouth of the silicone tip with your child's skin when cleaning nasal cavity. If the air inlet of the tip fits completely against the skin, a closed environment is created that prevents suction or reduces suction efficiency.
Difficulty removing discharge from the nasal cavity	<ul style="list-style-type: none">• Dry discharge is more difficult to remove. Before cleaning your child's nose, use saline to soften it and moisturize the nasal mucosa.
Nasal discharge comes out of the air outlet when cleaning the nasal cavity	<ul style="list-style-type: none">• Turn off the device and clean the collection cup. While cleaning your nose, monitor the level of secretions in the collection cup. It is recommended to empty and clean the collection cup when nasal discharge reaches half of cup's capacity. Too much discharge collected in the cup may cause clogging of the air pump and damage to the device.

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Product name	Feelo Family Baby Nasal Aspirator
Age group	From 0 to 12 years old
Maximum suction pressure	68 kPa
Modes	Gentle, moderate, strong
Input rated voltage	DC 5V 1A
Rated working voltage	3.7V
Rated working power	2W
Battery type	Li-ion battery
Working time	Up to 2-3 hours on a single charge (gentle mode)
Charging time	3 hours
Noise level	Approx. 65 dB
Dimensions	(D) 212 mm x (S)40 mm x (G)41 mm
Weight	168 g

WARNING! READ BEFORE USE

- For home use only.
- Do not use the nasal aspirator if you see wounds or signs of mucosal inflammation in your child's nose.
- In case of: fever, infection, irritation or nosebleeds, stop using the device and contact your general practitioner.
- Do not use this product for purposes other than intended. Do not put the aspirator in your ears, mouth or eyes.
- The nasal aspirator is a hygiene product. It is to be used only to clean a child's nose from naturally produced secretions.
- If your child feels unwell while using the device, stop using it immediately and consult a general practitioner.
- When using the nasal aspirator do not block the baby's other nostril to avoid headaches.
- Do not use excessive force when inserting the silicone tip into the nasal cavity.
- During operation, when the capacity of the collection cup reaches half, empty it before further use. This prevents mucus from entering the air pump and damaging the device.
- The device should be operated by adults. Do not allow children to play with the product.
- Before use, check that the device is well sealed and that the accessories are complete.
- Do not turn on the device while charging.
- The product is not intended to be repaired by user.
- Store the charging cable in a dry and safe place.
- Straighten the charging cable before use.
- In the event of a failure, do not dis-

semble the device and do not replace any components because there is a risk of electric shock. In event of failure contact the manufacturer.

- A damaged charging cable must be replaced with a new one. In event of failure contact the manufacturer.
- Do not expose the product to prolonged exposure to high temperature or sunlight. Do not place the nasal aspirator near heat sources.
- Do not microwave the nasal aspirator or its components.
- Always use a clean silicone tip to clean your nose.
- The nasal aspirator should be stored in a cool place.
- The nasal aspirator requires regular cleaning (see: Cleaning and storage). When washing do not immerse the entire device in water. Clean it with a damp and soft cloth. Wipe dry.

IMPORTANT WARNING!

- Turn off device RIGHT after snot sucking finished to avoid mucus back flow.
- If liquid enter machine, remove storage chamber then run machine for 5-10 minutes to drain liquid entered the motor. Wipe away stains afterwards.
- Empty and rinse the snot storage chamber when snot collected exceed 1/2 of whole capacity incase overflow.
- The nasal aspirator can NOT be turned on while charging.
- 1. Keep storage chamber and device CLEAN and DRY after use.
 2. To keep the battery healthy, charge the battery to 100% every two months if you do not need to use it for a long time.
 3. Please ONLY use 5V adapter, voltage over 5V can NOT charge the device.

WARNING

To negate the risk of electric shock:

- Always disconnect the device from the power source after use.
- Do not use while bathing.
- For more detailed instructions on the storage and use of the device and the included accessories, please contact the manufacturer.

To reduce the risk of ignition, short circuit or injury:

- After charging disconnect the charger cable from the electric socket.
- Product is not a toy. Be especially careful if the nasal aspirator is used by or in the presence of a child.
- Use the product in accordance with its intended purpose. We recommend using accessories recommended by the manufacturer.
- Do not use charger if the power cable

or plug is damaged. If the nasal aspirator does not work properly after being dropped or left outdoors return it to the point of purchase.

- Do not stretch the charger's power cord, cut it with a door, or wrap it around sharp edges. Do not turn on the nasal aspirator while charging. Keep device components away from heated surfaces.
- Do not disconnect the charger from the power source by pulling the cable. Use the plug.
- Do not hold the plug with wet hands.
- Never use the product if you have trouble concentrating or feel drowsy.
- Do not use the device outdoors or in places where it may be exposed to aerosols or pure oxygen.
- Plug the device into a power outlet. Check its specifications.

Children and adults with limited sensory,

motor or mental abilities should use the device under the supervision of a guardian. Children should not play with the device. Children should not clean or maintain the device without supervision.

TWO YEAR WARRANTY

The manufacturer provides a warranty for a period of two years from the date of purchase. The warranty is valid in Poland and on the premises of official distributors. The warranty covers damage resulting from manufacturing defects or incorrect workmanship of the product. Keep your proof of purchase for the warranty period. During the warranty period, you can submit a complaint if the product was used correctly and as intended.

The Warranty does not cover:

1) Surface damage resulting from normal

use.

2) Damage resulting from:

- a) accidental drop or impact, misuse, abuse or commercial use, negligence, modification of the device,
- b) improper maintenance, carelessness
- c) connecting to the inappropriate power source
- d) repairs or attempted repairs made by a party not authorized by the manufacturer. Any attempt to open or disassemble the device, its parts or accessories will void the warranty.

Warranty claims must be supported by evidence. Claim must be made within the warranty period. The material covering the handle is delicate, so be careful when using it. Scratches or abrasions on the surface of the handle will be treated as superficial damage resulting from normal use. Therefore, they will not be covered by the warranty. The warranty

applies to components that affect the functioning of the nasal aspirator. Proper care of the aspirator significantly increases its lifespan. Failure to follow the recommendations may result in the loss of the warranty.

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Made in P.R.C

Model: FF.AS.010.WHT.1
Input rated voltage: DC 5V 1A
Input rated power: 2W
Rated working voltage: 3,7V
Rated working power: 2W

The product meets the requirements of the so-called „New Approach” Directives of the European Union (EU).

This device is marked in accordance with the Polish Act on Waste Electrical and Electronic Equipment and the WEEE Directive 2012/19/EU with the symbol of a crossed-out waste bin. This marking indicates that this equipment, after its service life, cannot be disposed of with other household waste. The user is required to hand it over to those conducting the collection of waste electrical and electronic equipment. Those conducting the collection, including local collection points, stores, and municipal units, create an appropriate system that enables the disposal of this equipment. Proper handling of waste electrical and electronic equipment helps to avoid harmful health and environmental consequences arising from the presence of hazardous components and improper storage and processing of such equipment.

The symbol indicates Class II protection for the electrical device.

The packaging is recyclable.

Use caution when using the product.

IP22

IP (Ingress Protection) Class. The first digit (2) indicates protection against solid objects larger than 12.5 mm, and the second digit (2) indicates protection against water dripping at an angle of up to 15 degrees.

The product should be protected from moisture and stored in a dry place.

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